# LOOX

ENGLISH - INSTRUCTIONS FOR USE.  
PLEASE READ CAREFULLY MAKING  
SURE YOU FULLY UNDERSTAND THEM.

Please ensure that you read all the instructions in this leaflet before attempting to wear your new cosmetic lenses. By following these instructions you will get the most from your new look.

**Preparation** Preparing to wear hygiene is an essential part of preparation. Always wash and dry hands thoroughly, removing trace of anything that may cause contamination or damage, before handling your cosmetic lenses. Always soak your cosmetic lenses for at least 2 hours before first use in a suitable multipurpose soft contact lens solution.

**Achieving your look** Ensure your cosmetic lenses have soaked prior to first wear. Try to make a habit of putting on the right side first and always make a distinction between right and left. Place one lens on your index finger and check that it is clean and not inside out (ensure that it has a natural bowl like shape). Hold your upper lid open with your left hand's index and middle finger, at the same time hold your lower lid open with the middle finger of your right hand. Looking straight at your eye in the mirror, slowly and gently place the lens on your eye. Carefully release your eyelids and blink. You can gently massage your eyelid to centre the lens. When you are sure that it is correctly inserted, repeat the procedure for the other eye.

**Removing** Wash and dry your hands thoroughly. Look upwards and hold down the lower lid with the middle finger of your right hand. Placing your index finger and thumb of the same hand on opposite sides of the lens, slowly slide it down to the white of your eye. Gently pinch the lens from your eye, taking care not to pinch too hard. Repeat for the other eye.

**Wearing instructions** Cosmetic lenses are intended for occasional use only. Check with your eye care practitioner before wearing for prolonged periods on an everyday basis. Your eye care practitioner will recommend a suitable wearing schedule. It is suggested that you only wear your cosmetic lenses for 4 hours on the first day, increasing your wearing time by 2 hours daily until you reach optimum wearing time. If you have not worn your cosmetic lenses for several days, you should follow wearing schedule recommendations as a new wearer. Always rinse your lenses, before wearing and after removing, with a suitable multipurpose soft contact lens solution. After each wear, always store your cosmetic

lenses in fresh multipurpose soft contact lens solution making sure not to damage, or catch your lenses when storing in any lens case used. If you wear your cosmetic lenses infrequently, always remember to replace the solution used with fresh solution regularly, and ensure that the case used for storage of your accessories is cleaned daily. These cosmetic lenses have a maximum number of days use (see outer label) and are intended to be replaced at this time after opening, irrespective of the number of times worn.

**Safety** DO NOT wear if your cosmetic lenses appear to be damaged. DO NOT share your cosmetic lenses with anyone else. DO NOT let hairspray or any other cosmetic aerosols come into contact with your cosmetic lenses. DO NOT participate in any water sports whilst wearing. DO NOT wear your cosmetic lenses when using eye medication. DO NOT use tap water, saliva or any household cleaning agents to clean or wet your cosmetic lenses or storage case. ALWAYS remove your cosmetic lenses before you go to bed. ALWAYS put your cosmetic lenses in before applying any make-up. ALWAYS take out cosmetic lenses before removing any make-up. ALWAYS replace your cosmetic lenses if your eyes get red or irritated or if you have any discomfort or pain, and seek help from a qualified eye care practitioner.

**Important Information** Please note that these cosmetic lenses are not intended to correct any defect of sight and are intended only for cosmetic colour change. If you suffer from any of the following, then you MUST NOT wear these cosmetic lenses: corneal dystrophies, corneal ulcers, inflammation of the cornea, e.g. keratitis, inflammation of the eye, e.g. iritis, uveitis, allergic eye conditions, any pre-existing eye condition. If you are in any doubt, please seek further advice from medical or eye care practitioner. Cosmetic lenses are suitable for normal, healthy eyes. If you are in any doubt as to whether these lenses are suitable for you, then please consult a medical or eye care practitioner.

**Fitting and Aftercare** It is recommended that you consult your eye care practitioner to ensure suitability and for practical advice on wearing and removing (consultation fees may be charged).

115 First Street  
Suite 555  
Collingwood, ON  
L9Y 4W5 Canada  
1-888-755-5578 ext. 220

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